sepra/film®

ADHESION BARRIER

BARRIÈRE ANTI-ADHÉRENCES
 ADHÄSIONSBARRIERE
 ADHESIEBARRIÈRE
 ADHERENS BARRIÄR
 ADHÆRENCEBARRIÈRE
 ADHERANSEBARRIÈRE
 KIINNIKEMUODOSTUKSEN ESTÄJÄ
 BARRIERA ANTIADERENZE
 BARRERA ANTIADHERENCIA
 BARREIRA ANTI-ADERÊNCIA
 ΦΡΑΓΜΑ ΣΥΜΦΥΣΕΩΝ
 ADEZYON BARIYERİ

Genzyme Biosurgery
 76 New York Avenue
 Framingham, MA 01701 USA
 A division of
GENZYME CORPORATION
 500 Kendall Street
 Cambridge, MA 02142 USA
 Tel: 800-261-1570

Distributed in
 Canada by:

Genzyme Canada
 2700 Matheson Blvd. East
 Suite 800, West Tower
 Mississauga, ON
 L4W 4V9
 Canada
 General Inquiries:
 Tel. 877-220-8918
 Fax. 905-625-7811
 Medical and Product Complaints:
 Tel. 800-261-1570

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Distributed in
 Australia and
 New Zealand by:

Genzyme Australia
 Norwest Quay
 Level 2/Suite 202
 21 Solent Circuit
 Baukham Hills NSW 2153
 Australia
 Tel: +61-2-9680-8383
 Fax: +61-2-9634-2777

Importado e
 distribuído no
 Brasil por:

Genzyme do Brasil Ltda.
 Rua Padre Chico, 224 Perdizes
 05008-010 São Paulo SP
 CNPJ: 68.132.950/0001-03
 Reg. MS N°: 80149670003
 Farm. Resp.: Karina Zambotti de Brito CFR-SP: 22150
 SAC: 0800 77 123 73

Sepra Film and design is a registered trademark of
 Genzyme Corporation, Cambridge, MA, USA.

Genzyme is a registered trademark of
 Genzyme Corporation, Cambridge, MA, USA.

This product is covered by U.S. Patent Number 5,017,229

Genzyme Europe BV
 Gooimeer 10
 1411 DD Naarden
 The Netherlands
 Tel: +31-35-699-1200
 Fax: +31-35-694-3214

genzyme
 Biosurgery

CE 0088
 4564 (07/07)

DESCRIPTION

Seprafilm® Adhesion Barrier (Seprafilm) is a sterile, bioresorbable, translucent membrane composed of two chemically modified anionic polysaccharides, sodium hyaluronate (HA) and carboxymethylcellulose (CMC).

INDICATION

Seprafilm is intended as an adjunct in abdominal, pelvic, and thoracic surgery for reducing the incidence, extent and severity of postoperative adhesions at the site of placement.

INTENDED USE

Seprafilm should be applied to sites of potentially adhesiogenic tissue and organ structures in the abdominopelvic or thoracic cavity to serve as a temporary barrier separating opposing tissue surfaces.

WARNINGS

Seprafilm must be used according to the instructions for use. Read instructions prior to use. Seprafilm is supplied sterile and should not be resterilized. The membrane is for single use only. Every opened and unused Seprafilm pouch must be discarded.

Seprafilm should not be wrapped directly around a fresh anastomotic suture or staple line of the intestine. Clinical trial data on Seprafilm indicate that such use may result in an increased risk of anastomotic leak-related events (fistula, abscess, leak, sepsis and peritonitis). The incidence of these events was not affected when Seprafilm was placed elsewhere in the abdomen.

CAUTIONS

- Do not use if foil pouch is damaged or appears to have been tampered with or opened prior to time of use.
- The use of Seprafilm in combination with other adhesion prevention products has not been clinically evaluated.
- No controlled clinical studies have been conducted in patients with active infections.

Information for the Patient

- Foreign body reaction may occur as with most surgical adjuncts but have been rarely reported during clinical use.
- No pre-clinical reproductive studies have been conducted. No clinical studies have been conducted in women who become pregnant in the first month after application of Seprafilm. Therefore, avoiding pregnancy during the first complete menstrual cycle after the use of Seprafilm should be considered.

STORAGE INSTRUCTIONS

Seprafilm should be stored at 2-30°C.

HOW SUPPLIED

Seprafilm is packed in a Tyvek® holder within a plastic sleeve and packed in an outer, sealed, foil pouch. The contents of the foil pouch are sterilized by gamma radiation of 25-40 kGy.

Refer to package label for film size and quantity.

DIRECTIONS FOR USE

Preparation

- Open the foil pouch and drop the plastic sleeve on a dry, sterile field.
- Carefully remove the Tyvek® holder from the plastic sleeve.
- Keep membrane dry in the holder prior to application.
- While membrane remains in the holder, cut to desired size with scissors.
- Expose 2 cm of membrane through open end of the holder.

Placement

Apply Seprafilm immediately prior to closure of the abdominopelvic or thoracic cavity:

- Ensure surgical field is dry.
- Handle membrane carefully with dry instruments and/or gloves.
- Avoid contact with tissue surfaces until directly at site of application. If contact occurs, moderate application of a standard irrigating solution may be used to gently dislodge membrane from unintended tissue surfaces.
- When necessary, facilitate entry in abdominal, pelvic, or thoracic cavity by slightly curving the holder.
- Apply Seprafilm.
- Extend placement up to 8 cm beyond margins of direct trauma to achieve adequate coverage with the membrane.
- Let exposed membrane adhere to desired position on the tissue or organ by gently

pressing down with a dry glove or instrument, while withdrawing the holder.

- When necessary, moisten membrane lightly with irrigating solution (1-2 ml) to facilitate moulding of membrane along the tissue or around the organ contours.
- Up to 10 (13 cm x 15 cm) membranes per patient have been used in controlled clinical studies for the abdominopelvic indication, while up to 4 membranes per patient have been used in controlled clinical studies for the thoracic indication. Allow overlap of 2-3 cm between individual membranes to ensure coverage.

After placement

- Discard holder(s) following placement.
- Care should be taken not to disturb membrane after placement.
- Do not suture membrane in place.
- Close abdominopelvic or thoracic cavity according to standard surgical technique.

Seprafilm is a temporary bioresorbable barrier that is resorbed within one week and excreted from the body in less than 30 days. Therefore, no procedure is required to remove the membrane.

VIGILANCE

For adverse event reporting, contact:
 Genzyme Corporation
 500 Kendall Street
 Cambridge, MA 02142 U.S.A.
 Tel. 800-261-1570
 Fax. 617-761-8393

All others contact:
 Genzyme Europe B.V.
 Gooimeer 10
 1411 DD Naarden
 The Netherlands
 Tel. +31 (0)35 699 1299
 Fax. +31 (0)35 694 8756

In Canada, for general inquiries,
 you may contact:
 Genzyme Canada
 2700 Matheson Blvd. East
 Suite 800, West Tower
 Mississauga, ON
 L4W 4V9
 Canada
 Tel. 877-220-8918
 Fax. 905-625-7811

For medical and product complaints,
 please contact:
 Tel. 800-261-1570

For further information, please contact your local sales representative.

All others contact:
Genzyme Biosurgery
4620 Kingsgate
Cascade Way
Oxford Business Park South
Oxford
OX4 2SU
Tel: +44 (0)1865 405 200
Fax: +44 (0)1865 774 172

†Tyvek® is a registered trademark of the DuPont Company, Wilmington, DE, U.S.A.

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